

Work Flow Instructions for Phlebotomist

Please carefully review the following instructions.

Before Blood Draw

- ✓ Check the patient's photo ID to confirm that the Name and DOB match with what is written on the Test Requisition Form and Streck tubes.
- ✓ Ask the patient to confirm that the information printed on his/her Test Requisition Form and blood tubes is accurate.

Blood Draw

Follow the directions for use provided with the Cell-Free DNA BCT® tubes.

Size 20-23 gauge needles are recommended. Fill tubes as instructed on Streck Instructions for use. Preparation for Shipment after Blood Draw

- ✓ Write the date and time of collection on both tubes as well as on the Test Requisition Form.
- ✓ Place both tubes inside the provided biohazard bag, seal.
- ✓ The sample should be shipped immediately.
 - a. If the sample is not being shipped immediately, store at room temperature (6-37°C; 43-98°F) until shipping (do NOT refrigerate, freeze or ship on ice.)
- ✓ Remove the FedEx Clinical Pak from the shipping box.
- ✓ Place the biohazard bag and completed paperwork into the shipping box.
- ✓ Seal the shipping box by removing the adhesive strip on the lip and inserting the side flaps into the side panels. Push down firmly to ensure it is sealed.
- ✓ Place the shipping box into the FedEx Clinical Pak and seal.
- ✓ Call 1-800-GOFEDEX (1-800-463-3339) schedule a free pickup, or drop off the package at your nearest FedEx service center or FedEx dropbox.

Please contact the Pathway Genomics Client Services Department at (877) 505-7374 or clientservices@pathway.com with any questions.



CLIA No. 05D1092505 | CAP No. 7232232



For more information about Pathway Genomics Corporation
www.pathway.com | 877.505.7374 | clientservices@pathway.com

